
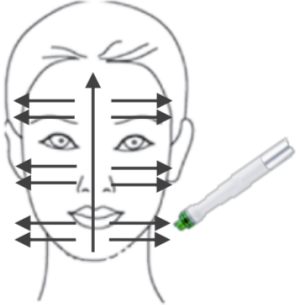

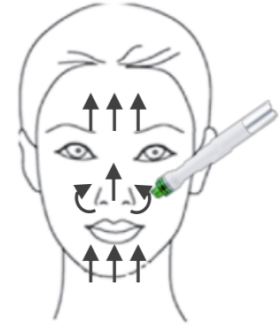



## ProFacial Premium multi-dimensional AQUA HYDRA FACIAL PEELING

 <p><b>PS 1</b> The first step for clear dead skin cells removal</p>	<p>Contains Lactic Acid-AHA</p> <ul style="list-style-type: none"> <li>• For the use on normal skin</li> <li>• Exfoliate and remove impurities from deep inside the pore</li> </ul> <p>VACUUM LEVEL : Medium TX: 5 min</p> <p>Lightly apply the solution on the face and brush two strokes horizontally to spread out evenly (the tip should be completely adhered to the skin during the brushing). Carry out the treatment from down to up.</p>	
 <p><b>PS2</b> The first step for intensive skin pore care &amp; acne</p>	<p>Contains Salicylic Acid-BHA</p> <ul style="list-style-type: none"> <li>• For the use on acne prone skin</li> <li>• Emulsify and carry away sebum for acne prone skin, comedonal extraction and blackhead treatment</li> </ul> <p>VACUUM LEVEL : Strong TX: 5 min</p> <p>Start from the nose and round the nose. When there is a curve, make a short round cut. Lightly apply when treating sensitive area like forehead, philtrum and chin area. Each area of treatment should not exceed a minute.</p>	
 <p><b>PS3</b> The second step for intensive absorption of active ingredients &amp; moisturizing</p>	<p>Contains highly enriched essence with EGF</p> <ul style="list-style-type: none"> <li>• To be used after treating with PS1 or PS2</li> <li>• Enhance hydration with hyaluronic acid, EGF and nutrients</li> </ul> <p>VACUUM LEVEL : Medium TX: 5 min</p> <p>When the solution starts to flow, adjust the vacuum and flow level accordingly. Lightly apply the solution on the face and brush two strokes horizontally to spread out evenly (the tip should be completely adhered to the skin during the brushing). Carry out the treatment from down to up.</p>	